



The image shows a Fisher & Paykel MR810 Respiratory Humidification System. It consists of a white base unit with a control panel on the front. The panel includes a power button, a volume knob, a fan speed indicator with three yellow LEDs, and a warning light. On top of the base is a clear plastic humidifier chamber with a blue corrugated tube connected to it. The background is a blurred hospital room with a patient in a bed.

Inspired Care

The simple solution
for humidification

*MR810 Respiratory
Humidification System*

Fisher & Paykel
HEALTHCARE


Respiratory
HUMIDIFICATION

MR810 Respiratory Humidification System



MR810 HUMIDIFIER

- Two simple control buttons

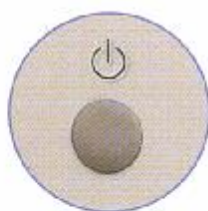

Selects on/off




Selects one of three temperature and humidification levels. The level can be adjusted for the patients comfort.



CIRCUIT

- **Heater wire mode indicator**
 - when the green light is on, the circuit is connected correctly.

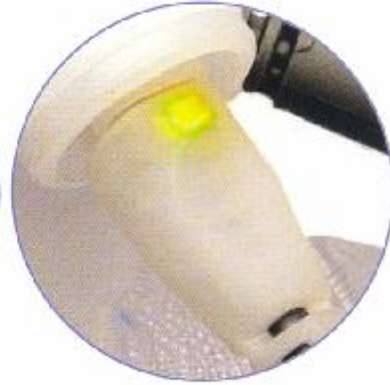

CHAMBER

- **MR290 chamber**
 - automatic waterfeed line
 - delivers constant humidification
 - no need to manually refill chamber
- **Unique dual float mechanism**
 - safely maintains water level
- **Suitable for all patients**
 - neonates, paediatrics and adult applications



- **Unique spiral heater wire**
 - provides even circuit heating
 - Ambient temperature sensor increases the humidity performance by minimizing condensate in the circuit



The MR810 Respiratory Humidification System

MR810

- **Supply voltage:** 230 V~, 115 V~ or 100 V~
- **Frequency:** 50/60 Hz
- **Maximum power usage:** 200 W
- **Flow range:** 5 L/min - 60 L/min
- **Warm-up time:** 30 minutes

MR290 Autofeed Chamber

Single Patient Use Circuits

- **RT Series**
- **Inspiratory heated breathing circuits**

Warning: The use of breathing circuits, chambers or other accessories with the MR810 humidifier which are not approved by Fisher & Paykel Healthcare may impair performance or compromise safety

INTERNATIONAL P.O. Box 14 348, Panmure, Auckland 1134, **New Zealand** Tel:+64 9 574 0100 Fax:+64 9 574 0158

Email: info@fphcare.com **USA** Tel: 1800 446 3908 or +1 949 470 3900 Fax:+1 949 470 3933

AUSTRALIA Tel:+61 3 9879 5022 Fax:+61 3 9879 5232 **AUSTRIA** Tel: 0800 29 31 23 Fax: 0800 29 31 22

BENELUX Tel:+31 78 644 0924 Fax:+31 78 644 0914 **CHINA** Tel:+86 20 8363 4286 Fax:+86 20 8363 3595

FRANCE Tel:+33 1 6446 5201 Fax:+33 1 6446 5221 **GERMANY** Tel:+49 7182 93777 0 Fax:+49 7182 93777 99

INDIA Tel:+91 80 51236041 Fax:+91-80-51236044 **IRISH REPUBLIC** Tel: 1800 409 011

ITALY Tel: +39 06 7839 2939 Fax:+39 06 7814 7709 **JAPAN** Tel:+81 3 3661 7205 Fax:+81 3 3661 7206

SPAIN Tel:+34 902 013 346 Fax:+34 902 013 379 **UK** (EU Authorised Representative) Tel:+44 1628 626 136 Fax:+44 1628 626 146

Nthn Ireland Tel: 0800 132 189

www.fphcare.com

Respiratory

HUMIDIFICATION

Fisher & Paykel
HEALTHCARE

REF 185042818 Rev B 2004-07 